419611C

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¹Etude Challenge: Meunier S, et al. A randomized, controlled, double-blind prospective trial with a Lipido-Colloid Technology/Nano-Oligo Saccharide Factor wound dressing in the local management of venous leg ulcers. Wound Rep Reg. 2012.

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Made in France
Laboratoires URGO - 42 rue de Longvic - 21300 CHENOVE - France

EN SOFT-ADHERENT FOAM DRESSING WITH TLC-NOSF.

DESCRIPTION

UrgoStart is a lipido-colloid foam dressing hydrocellular type with soft-adherent TLC-NOSF, an innovative Technology developed by the Laboratoires Urgo.
UrgoStart is composed of:
- a soft-adherent TLC-NOSF layer combined with an absorbent polyurethane foam pad
- a protective non-woven polyurethane backing

PROPERTIES

UrgoStart offers all the benefits of TLC-NOSF (moist environment, faster wound healing).
TLC-NOSF
TLC (Technology Lipido-Colloid) combined with NOSF (Nano-Oligo Saccharide Factor) is a patented innovative technology which in contact with wound exudate forms a gel and creates a moist environment, enabling the key cells involved in the repair process (fibroblasts, keratinocytes, macrophages) to exert their action.
TLC-NOSF interacts with the micro-environment of the wound by preventing the detrimental effect of Matrix MetalloProteases (MMPs) which in excess in chronic wounds creates a continuous degradation of the extracellular matrix and delayed wound healing.
TLC-NOSF restores balance in the wound bed so granulation of chronic wounds can occur. **UrgoStart** thereby promotes faster wound healing.
The efficacy of TLC-NOSF has been demonstrated in several clinical studies including Randomised Controlled Trials (Challenge study¹).
The absorbent polyurethane foam pad of **UrgoStart** ensures high fluid management, preventing maceration.
The backing of **UrgoStart** is flexible, conformable and very comfortable : it allows a good conformability to the various anatomical relief of the wound.

UrgoStart facilitates nursing care regarding ease of use and patient compliance.
It is suitable for use under compression bandaging when prescribed, due to its ability to retain exudate.

The properties of **UrgoStart** are numerous:

- accelerates wound healing
- absorption of exudate and drainage preventing any risk of maceration
- maintains a moist environment favorable to the wound healing process
- very conformable and easy to reposition

¹Challenge Study: Meunier S, et al. A randomized, controlled, double-blind prospective trial with a Lipido-Colloid Technology/Nano-Oligo Saccharide Factor wound dressing in the local management of venous leg ulcers. Wound Rep Reg. 2012.

INDICATIONS

UrgoStart is indicated for the treatment of exuding chronic wounds (leg ulcers, pressure ulcers, diabetic foot ulcers), long standing acute wounds.
The **UrgoStart** heel version is recommended for the treatment of chronic exuding wounds located in the heel area (heel pressure ulcers...).

INSTRUCTIONS FOR USE

- Preparation of the wound:
 - Clean the wound as per local protocol.
 - If an antiseptic is first used, carefully rinse the wound with normal sterile saline before applying **UrgoStart**.
 - Dry carefully the surrounding skin.
 - **UrgoStart** can be cut using sterile equipment to adjust the size of the dressing to fit the wound.
- Application of the dressing:
 - **Standard version**
 - Remove the protective wings.
 - Apply the soft-adherent side of **UrgoStart** to the wound.
 - Secure the dressing in place with a suitable bandage.
 - Apply compression bandaging over the dressing when prescribed.
 - **Heel version**
 - The dressing is an arrow shape. Remove the protective wings.
 - Place the foot pointing in the direction of the arrow with the heel in the centre of the soft-adherent TLC side (narrow part).
 - Fold the back of the dressing up and around the back of the heel. Mold the side edges up and around the foot.
 - Secure the dressing in place with a suitable bandage.
 - Dressing changes:

UrgoStart dressing may be changed every 2 to 4 days, and up to 7 days depending on the exudate level and the clinical condition of the wound.
The recommended treatment duration is a minimum of 8 weeks.

PRECAUTIONS FOR USE

- The soft-adherent TLC layer of **UrgoStart** adheres to surgical latex gloves. It is therefore recommended to avoid contact with this soft-adherent TLC layer or by using sterile clamps.
- If signs of critical colonisation are present, suitable treatment is recommended prior to the use of **UrgoStart**.
- In case of an atypical ulcer presenting induration or overgranulation, **UrgoStart** should only be used after checking the absence of wound-related malignancy in order not to delay the diagnosis.
- The retriggering of the healing process may possibly cause stinging or painful sensations on commencement of treatment with **UrgoStart**. This rarely warrants suspension of treatment.
- In the absence of clinical data as a first line treatment in acute wounds, and in the treatment of Epidermolysis bullosa (even for longstanding lesions), it is not recommended to use **UrgoStart**.
- **UrgoStart** must not be used in a hyperbaric chamber.
- Single use sterile individual packaging: re-using a single use dressing may lead to risks of infection.
- Do not re-sternilise the dressing.
- (*) Check that the sterile protector is intact before use. Do not use if package is damaged.

CONTRA-INDICATIONS

In order not to delay any optimal treatment, **UrgoStart** is contra-indicated in wounds which are not likely to heal such as:

- Cancerous wounds,
- Fistula wounds which may reveal a deep abscess.

Do not use **UrgoStart** if there is a known sensitivity to the dressing.

Made in France
Laboratoires URGO - 42 rue de Longvic - 21300 CHENOVE - France

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Dear Madam, dear sir,
Thanks to send us 5 sets of triptych
concerning this item, Best regards.